



NYLON PLASTIC COMPONENTS - MECHANISM & LEVERS

For cleaning

Use soft cloth and non-abrasive neutral detergent. Do not use alcohol, solvent-based, acid or abrasive cleaners.

For disinfection

We recommend the use of sodium hypochlorite (0.1% - 0.5% sol.) or hydrogen peroxide.*



POLYPROPYLENE PLASTIC COMPONENTS - SEAT

For cleaning

Use soft cloth and non-abrasive neutral detergent. Do not use alcohol, solvent-based, acid or abrasive cleaners

For disinfection

Denatured alcohol, sodium hypochlorite (0.1% - 0.5% sol.), ethanol (70% sol.), hydrogen peroxide (0.5% sol.) and benzalkonium chloride can be used.*

PRODUCT NAME

Epsilon Integrated seat

TYPE

Weight activated mechanism

MATERIALS

Nylon plastic

Polypropylene plastic

Before disinfecting the entire surface of any component, it is advisable to proceed with tests on a small part of the product.

* Analysis of the effects resulting from the application of chemicals on product surfaces, performed by an accredited laboratory.